

# SUREHOLD SAFETY SCALPEL HANDLE



The Avantik SureHold Safety Scalpel Handle reduces the risk of injury from scalpel malfunction and user errors. How often has a scalpel blade popped off? How often have you injured yourself while installing a scalpel blade using force? With the Avantik SureHold Safety Scalpel Handle, there is no force needed to install the blade. The blade will not pop off under pressure and you will be safer and your lab will be more efficient for using it.

## Features

- Pivoting blade handle
- Blade locking mechanism
- Easy and safe blade replacement

## Benefits

- Blade is easily attached
- Decrease the risk of getting cut
- Blade will not pop off under pressure

## Product Information

**GR2144-A** Fits Scalpel Blades #10-#15

**GR2144-B** Fits Scalpel Blades #20-#25, #60, #70

Call us today.

**800-783-9424**  
**info@avantik-us.com**  
**avantik-us.com**



# SUREHOLD SAFETY SCALPEL HANDLE

## INSTRUCTIONS



### ATTACH A BLADE TO THE AVANTIK SUREHOLD SAFETY SCALPEL HANDLE:

**Step 1.** Open the SureHold Safety Scalpel Handle by releasing the top from the SureHold Pin.

**Step 2.** Open the handle to reveal the blade receptacle.

**Step 3.** Be sure the blade is the right size for your SureHold Safety Handle. Place the new blade securely onto the receiving pins.

**Step 4.** Close the handle and lock it into place using the pin to secure the handle and blade in place. Your scalpel is now ready to use.



### REMOVE & DISPOSE OF A BLADE FROM THE AVANTIK SUREHOLD SAFETY SCALPEL HANDLE:

**Step 1.** Open the SureHold Safety Scalpel Handle by releasing the top from the SureHold Pin.

**Step 2.** Hold the handle over a sharps container

**Step 3.** Turn the Avantik SureHold Safety Scalpel Handle over to release the used blade into the container.

Call us today.

800-783-9424

info@avantik-us.com

avantik-us.com

