

# QLICKSMART BLADEFLASK EVO

## THE LATEST INNOVATION IN SCALPEL SAFETY

The **Qlicksmart BladeFlask EVO** provides flexibility in your choice of scalpels and confidence to their removal with an easy to use single step removal process. Embrace safety with the new standard in scalpel blade removal.



# NEW!

### Qlicksmart BladeFlask EVO

**Item #:** GR2214 | **Qty:** 1/Each or 24/Case

**Item #:** GR2218 | **Qty:** 1/Mount Bracket

## Product Specifications


<b>Blade Capacity</b>	100 Blades
<b>Compatible Blades &amp; Handles</b>	Scalpel Blade Sizes 6-18, 20-27, 60 & 70 Scalpel handles including PM6 and PM8
<b>Size (per unit)</b>	9cm x 13.4cm x 11.3cm
<b>Accessories</b>	Mounting Bracket - holds the BladeFlask EVO in place to allow single-handed blade removal
<b>Material</b>	<b>Body:</b> Acrylonitrile Butadiene Styrene (ABS), <b>Mechanism:</b> ABS + Stainless Steel
<b>Reusable</b>	<b>Multi-use:</b> Each unit removes 100 scalpel blades before shut off
<b>Disposal Method</b>	The infectious substance logo on the front label and the correct sharps container color of the unit allow it to be disposed of as medical waste. Please follow your establishment's procedures for the disposal of sharps medical waste

Contact Us Today!  
[avantik-us.com](http://avantik-us.com)  
800.783.9424



# Mount



To Wall or Trolley

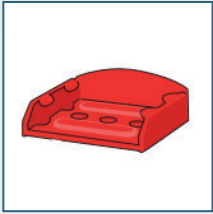

1. If you haven't mounted your BladeFlask EVO before, prepare the universal Mounting Bracket\*.

- ❖ The bracket has double-sided tape on the bottom, or use screws.

\*See fitting template and instructions inside the bracket packaging.

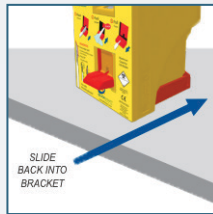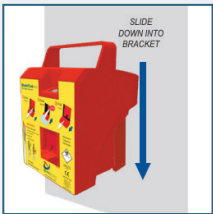


2. BladeFlask EVO can be mounted on a wall, trolley, or bench.


- ❖ **NOTE:** BladeFlask EVO should be mounted on the Mounting Bracket during use.

# Remove Blade


Single-Handed Action

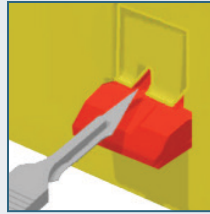

1. **Place** blade on the landing pad with the sharp edge facing to the right.

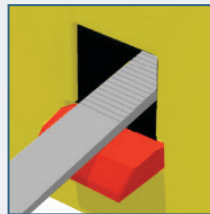

2. **Push** up the scalpel handle into the BladeFlask EVO along the ramp until you hear a “Click”.

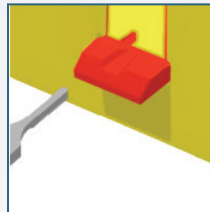

3. **Remove** the handle.



**NOTE:** BladeFlask EVO is disposed of as medical waste after the counter automatically shuts off when it reaches full capacity.



## SCAN NOW

to order the  
Qlicksmart BladeFlask EVO

Contact Us Today!

[avantik-us.com](http://avantik-us.com)  
800.783.9424

MKTG-SS-P0014

