


Instructions

Attaching, Removal and Disposal of Blades from the TrimSafe Handle.

TrimSafe HANDLE & BLADES

PATENT PENDING

Item# GR2158-A or B TrimSafe Handle 130 or 260mm Item# GR2163-A or B TrimSafe Blades 130 or 260mm

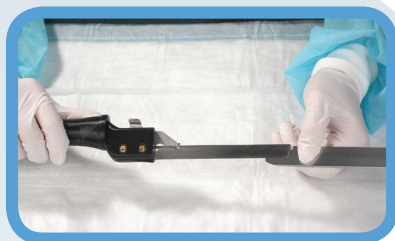

Attaching a TrimSafe Blade to the TrimSafe Handle:

Step 1. Carefully remove a TrimSafe Blade from the package, and insert the blade with the rounded edge towards the handle first.

Step 2. Slide the TrimSafe Blade back towards the handle base.

Step 3. Depress the thumb lever on the Handle in order to allow the blade to sit all the way back to meet with the TrimSafe Handle's edge.

Step 4. Slightly depress the thumb lever to allow the blade to fully rest against the base of the handle and lock it safely in place with the release of the thumb lever latch. Your TrimSafe Handle and Blade are now ready for use.



Removal & Disposal of a TrimSafe Blade:

Step 1. Use a proper sharps container to fit the length of your blade.

Step 2. Depress the thumb lever to create a "gap" between the TrimSafe Handle and the TrimSafe blade.

Step 3. Insert the blade and handle into the sharps container and using the edge of the sharps container to catch the "gap", slide the blade off the handle and allow it to drop into the sharps container without having to touch the blade.

Visit www.avantik-us.com/TrimSafe to view our instructional video.

For more information, please visit our website or call!

[avantik-us.com](http://www.avantik-us.com)

800.783.9424

